


## ROHO® SELECT® Series Cushion Sizing Chart for:

## CENTIMETERS

- ROHO QUADTRO SELECT® LOW PROFILE® Cushion (QS)
- ROHO QUADTRO SELECT® MID PROFILE™ Cushion (QS)
- ROHO QUADTRO SELECT® HIGH PROFILE® Cushion (QS)
- ROHO CONTOUR SELECT® Cushion (CS)

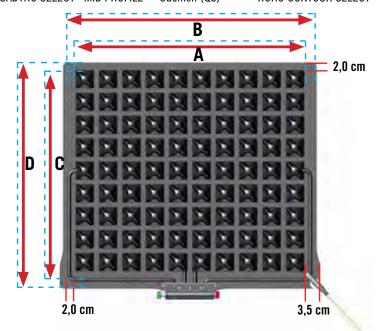

SELECT Cushions (QS and CS) "Fits Chair" Range CM									
Width (CM)					Depth (CM)				
Measured Chair Range (narrowest to widest)		Number of Cells	<b>A</b> Cushion Cell to Cell	<b>B</b> Cushion Base	Measured Chair Range (narrowest to widest)		Number of Cells	<b>C</b> Cushion Cell to Cell	<b>D</b> Cushion Base
21,0	25,5	5	21,0	28,0	21,0	25,5	5	21,0	25,5
25,5	29,5	6	25,5	32,5	25,5	29,5	6	25,5	30,0
29,5	34,0	7	29,5	36,5	29,5	34,0	7	29,5	34,5
34,0	38,0	8	34,0	41,0	34,0	38,0	8	34,0	38,5
38,0	42,5	9	38,0	45,0	38,0	42,5	9	38,0	43,0
42,5	47,0	10	42,5	49,5	42,5	47,0	10	42,5	47,0
47,0	51,0	11	47,0	54,0	47,0	51,0	11	47,0	51,5
51,0	55,5	12	51,0	58,0	51,0	55,5	12	51,0	56,0
55,5	60,0	13	55,5	62,5	55,5	60,0	13	55,5	60,0
60,0	64,0	14	60,0	67,0	60,0	64,5	14	60,0	64,5

Cell to cell and edge to edge dimensions are rounded to the nearest 0,5 cm.

The ISOFLO MEMORY CONTROL® will extend past the front edge of SELECT cushions.

Please check price list to confirm that a size is available. Sizes vary per model.

## Select the correct ROHO SELECT® Series Cushion for your wheelchair:

Step 1 Measure the wheelchair seat width and depth. Note: Wheelchair manufacturers' published seat dimensions may vary.

Step 2 Use your wheelchair seat dimensions and the table above to help you determine the number of cells in cushion width and in depth. The current version of this table is available at www.roho.com.

**Step 3** Select cushion cell height: HIGH PROFILE = 10,5 cm

MID PROFILE = 8,5 cm LOW PROFILE = 5,5 cm

CONTOUR SELECT = various cell heights

