




# Standard DRY FLOATATION® Cushion Sizing Chart for:

**CENTIMETERS**

- ROHO LOW PROFILE® Single (1R) or Dual (2R) Compartment Cushion
- ROHO HIGH PROFILE® Single (1R) or Dual (2R) Compartment Cushion
- ROHO MID PROFILE™ Single (1R) or Dual (2R) Compartment Cushion
- ROHO ENHANCER® Cushion (ENH)

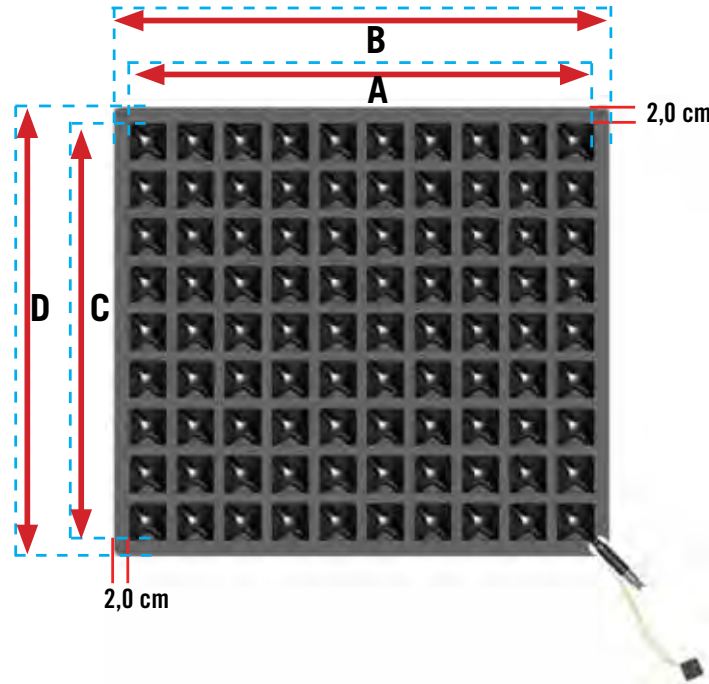

STANDARD Cushions (1R / 2R / ENH) “Fits Chair” Range CM									
Width (CM)					Depth (CM)				
Measured Chair Range (narrowest to widest)		Number of Cells	A Cushion Cell to Cell	B Cushion Base	Measured Chair Range (narrowest to widest)		Number of Cells	C Cushion Cell to Cell	D Cushion Base
21,0	25,5	5	21,0	25,0	21,0	25,5	5	21,0	25,0
25,5	29,5	6	25,5	29,0	25,5	29,5	6	25,5	29,0
29,5	34,0	7	29,5	33,5	29,5	34,0	7	29,5	33,5
34,0	38,0	8	34,0	37,5	34,0	38,0	8	34,0	37,5
38,0	42,5	9	38,0	42,0	38,0	42,5	9	38,0	42,0
42,5	47,0	10	42,5	46,5	42,5	47,0	10	42,5	46,5
47,0	51,0	11	47,0	50,5	47,0	51,0	11	47,0	50,5
51,0	55,5	12	51,0	55,0	51,0	55,5	12	51,0	55,0
55,5	60,0	13	55,5	59,5	55,5	60,0	13	55,5	59,5
60,0	64,0	14	60,0	63,5	60,0	64,5	14	60,0	63,5
64,0	68,5	15	64,0	68,0	64,0	68,5	15	64,0	68,0

Cell to cell and edge to edge dimensions are rounded to the nearest 0,5 cm.

Please check price list to confirm that a size is available. Sizes vary per model.

## Select the correct ROHO DRY FLOATATION® Cushion for your wheelchair:

**Step 1** Measure the wheelchair seat width and depth. Note: Wheelchair manufacturers’ published seat dimensions may vary.

**Step 2** Use your wheelchair seat dimensions and the table above to help you determine the number of cells in cushion width and in depth. The current version of this table is available at [www.roho.com](http://www.roho.com).

**Step 3** Select cushion cell height: HIGH PROFILE = 10,5 cm      LOW PROFILE = 5,5 cm  
MID PROFILE = 8,5 cm      ENHANCER = various cell heights

